**nosko**<sup>®</sup>  
natural baby & kids care

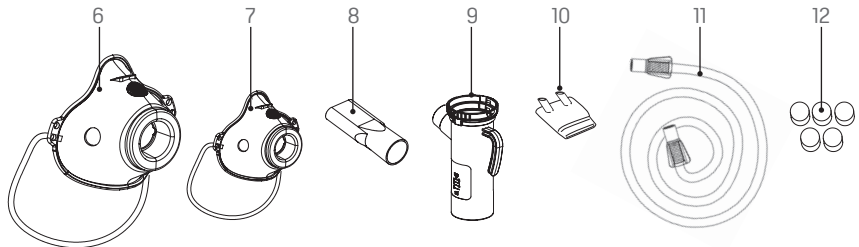
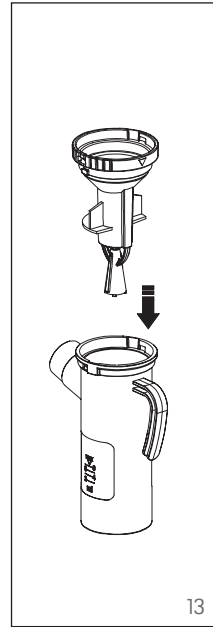
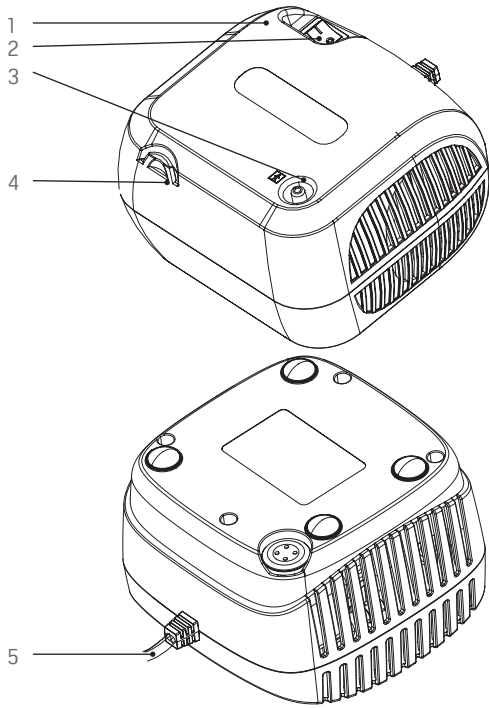
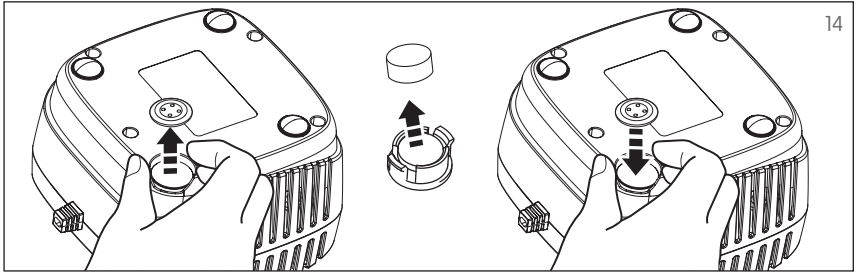
# Aerosol AIR 100 Family Compressor nebulizer



10756

PIC. 1 / AFB. 1 / FIG. 1 / АBB. 1 / PIC. 1 / OBR. 1 / OBR. 1 / OBR. 1 / ÁBRA. 1 / Фиг. 1

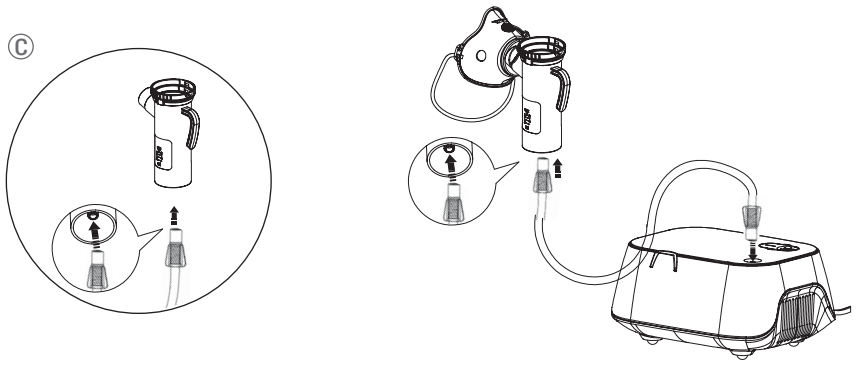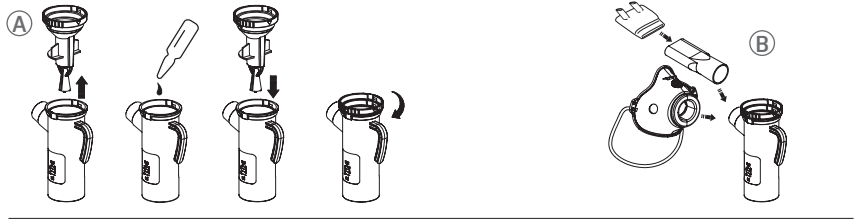**PIC. 2 / AFB. 2 / FIG. 2 / ABB. 2 / PIC. 2 / OBR. 2 / OBR. 2 / OBR. 2 / ÁBRA. 2 / Фиг. 2**  
 Short instructions / korte instructies / Instructions courtes / kurze Anleitung / rövid utasításokat / stručné pokyny / stručné pokyny / kratka uputstva / scurte instrucțiuni / кратки инструкции

# Aerosol AIR 100 Family

## Compressor nebulizer

EN

1. Compressor
2. ON/OFF button
3. Air tube outlet
4. Nebulizer holder
5. Power cable
6. Adult mask
7. Pediatric mask
8. Mouthpiece
9. Nebulizer
10. Soft nose piece
11. Air tube
12. 5 Air filters
13. Components and assembly of nebulizer
14. Air filter replacement

**Read the instructions carefully before using this device.**

**BF type part applied.**

Dear Customer,

This device is a high-quality aerosol therapy system, ideal for home use. It is provided with the nebulizerkit, which has been designed to treat respiratory illnesses quickly and effectively. It can be used with the most common aerosol-therapy medicines.

## INTENDED USE

---

This product has been designed for the inhalation of medicines for the treatment of respiratory illnesses.

### Intended audience

- Certified medical professionals (physicians, nurses and therapists).
- Patients in their home environment with home health-care support for treatment under the directions of a qualified physician.

The user must understand the general operation of the device and the content of the instruction manual.

It is forbidden to use the device on patients who are unconscious or who cannot breath on their own.

### Intended patients

People who suffer from respiratory illnesses and who need to take inhalation medicines. Suitable both for pediatric and adult use.

### Environment

This product has been designed for home use as well as for medical clinic settings and hospital use.

### Minimum product lifespan

The minimum product lifespan periods are reported below and are based on the assumption that the product is used to nebulize a saline solution (2ml) twice a day, 7.5min at a time, at an ambient temperature of 23°C.

The lifespan period may vary depending on the usage environment.

- Compressor (main unit): 3 years
- Nebulizer - air tube - mouthpiece - nose piece - pediatric mask - adult mask: 1 year. Frequent product use may shorten the expected lifespan.
- Air filter: 60 days.

### Warnings

Follow the warnings and precautions reported in the instruction manual.

## IMPORTANT PRECAUTIONS

---

- Follow the instructions for use. This manual contains important information on the operation and safety of this device. Please, read the instructions carefully before using the device and keep them handy for future reference.
- This device must be used exclusively as described in this manual. The manufacturer shall not be liable for any damages caused by improper use.
- The device is not suitable for anesthesia and pulmonary ventilation.
- The device must be used only with its original accessories as listed in the user manual.
- Do not use the device if you deem it damaged or if you notice any anomaly.
- Never open the device.
- This device has been manufactured using sensitive components and it must be handled with care.
- Please follow the storage and operation instructions detailed in the "Technical Specifications" chapter.
- Make sure to safeguard the product from:
  - water and humidity
  - extreme temperature
  - impacts and falls
  - contamination and dust
  - direct sunlight
  - heat and cold
- Use this device only with the medicines prescribed by the physician and follow dosing, duration and frequency instructions for the prescribed therapy.
- Do not bend the nebulizer beyond 45°.
- Do not use the device close to strong magnetic fields such as cellular phones and radio installations.
- Keep a minimum distance of 3.3 m from other equipments when using this device.
- Store the device and its accessories in a clean place.
- Do not use and store this device if smoke or gas is present.
- Do not use any medicines left in the nebulizer. Always use a fresh new dose.
- Do not use this product on anesthesia or pulmonary ventilation circuits.

Children must always use this device under adult supervision. Some parts are small and may represent a choking hazard if swallowed. Cables and tubes may represent a choking hazard.

Electronic devices must be disposed of according to local regulations and not as ordinary household waste.

The use of this device does not replace the medical advice of a physician.

---

## HOW TO PREPARE AND USE THE DEVICE

---

When using the device for the first time, make sure to clean all parts as described in the section "Cleaning and Disinfection".

1. Open the nebulizer (9) and pour in the amount of medicine as prescribed by the physician. Make sure not to pass the maximum level mark (Pic. 2/A).
2. Choose the most suitable inhalation accessory, as prescribed by the physician (Pic. 2/B).
3. Connect the nebulizer to the compressor using the air tube (Pic. 2/C) and plug in the power cord in an electric outlet making sure that the supply voltage is in compliance with what is stated in this manual or the device nameplate.
4. Press the ON/OFF (2) button to turn on the device. Place the mouthpiece in the mouth (8), let the nose piece rest (10) under the nostrils or the mask (6 or 7) on the face making sure to cover mouth and nose.
5. Sit in a relaxed position with the upper body straight.  
**Do not lie down while inhaling.**
6. After having completed the therapy session as recommended by the physician, press the ON/OFF (2) button to turn off the device and unplug the power cord from the electric outlet.
7. Make sure that the air tube contains no condensation. In case of condensation, switch on the device to dry up any moisture in the tube and then disconnect it as described above.
8. Empty out the remaining medicine from the nebulizer and clean the device in all of its parts, as described in the section "Cleaning and Disinfection".

 This device has been designed to work 20 min. ON / 40 min. OFF.

 Do not cover the device with a cloth, towel or anything else when in operation to avoid overheating or malfunction of the device.

## CLEANING AND DISINFECTION

---

Carefully clean all parts and remove any medicine residues and possible impurities after each treatment. The compressor (1) and the air tube (11) must be wiped clean with a clean damp cloth.

 Always wash hands thoroughly before cleaning and disinfecting the accessories.

 Wash/sterilize every part of the nebulizer.

 Do not expose the compressor to water or heat.

 Replace the air tube when passing to a new patient or in case of impurities.

 **Before cleaning, disconnect the plug from the electric outlet.**

 **The compressor is not waterproof and must not get in contact with liquids. In case of contact with liquids, immediately disconnect the power cord, shut off the device and dry it with care. Do not immerse the device in liquids.**

### Cleaning with water

- Clean all of the device's parts (**except for the air tube**) under hot running water for about 5 minutes, adding, a small amount of neutral detergent if necessary, following the dosage and limits specified by the detergent manufacturer.
- Rinse thoroughly, taking care to remove all residues, then leave to dry.

### Disinfection

- All parts (**except for the air tube**) can be disinfected with chemical disinfectants, using doses and limits as specified by the manufacturer of the disinfectant.
- Disinfectants are generally available from pharmacies.

### Steam sterilization

- All of the components (**except for the air tube and the masks**) can be steam-sterilized up to a temperature of 100°C (15-20 min.).
- After sterilizing, always leave the components to cool down to ambient temperature before using again.

 Do not repeat the sterilization cycle when the components are still hot.

### Sterilization in boiling water

- All of the components (**except for the air tube and the masks**) can be immersed in boiling water for no longer than 1 minute.

-  Boil the components in plenty of water.
-  Disassemble all components of the nebulizer and boil them separately.

## MAINTENANCE, STORAGE AND SERVICING

---

- It is recommended to replace the nebulizer, mouthpiece, nose piece and masks after 1 year of use.
  - Do not dry the nebulizer in the oven or with a hairdryer or its components may be damaged.
  - Always check that the filter is clean and replace it immediately if dirty or after a maximum of 60 days of use. Spare filters are provided with the device.
  - To replace the filter (14), remove the filter cover from its seat.
  - Remove the used air filter and insert the new filter. Reposition the filter cover in its seat.
  - Do not use the device without a filter installed. Use only original filters.
  - Do not wash the filter. If the filter is wet, replace it. If the filter is damp it may cause the device to stop operating.
  - Wash the filter cover regularly (do not boil it) then dry it carefully before repositioning it in its seat.
-  Never open the compressor compartment.
-  Repairs must be performed exclusively by specialized personnel.

## MALFUNCTIONS AND TROUBLESHOOTING

---

### The device does not turn on

- Make sure that the plug is well inside the wall socket.
- Make sure to have pressed the ON/OFF (2) button.
- Make sure that the device is operating within the limits established in this instruction manual (20 min. ON / 40 min. OFF).

### Nebulization is weak or not present

- Make sure that the air tube (11) is correctly connected at both ends.
- Make sure the air tube is not squashed, bent, soiled or blocked. If necessary, replace with a new one.
- Make sure the nebulizer is completely assembled (13) and ensure the components are correctly assembled.

- Make sure the medicine has been poured into the nebulizer.
- Make sure the filter is clean and the filter cover is positioned correctly.

#### **The device is noisy**

- Make sure to have inserted the filter correctly in its seat (14).

**Do not attempt to repair the device.**

## **WARRANTY**

---

This device has a **3-year** warranty from the date of purchase. The warranty is valid only with the original purchase receipt.

- The 3-year warranty is valid only for the compressor.
- Opening or tampering with the device voids the warranty.
- The warranty does not cover damage caused by improper use, accidents or failure to abide by the instructions for use.

## TECHNICAL DATA

---

<b>Max filling volume:</b>	8 ml
<b>Min filling volume:</b>	2 ml
<b>Residual volume:</b>	0.5 ml
<b>Neb rate 1 min (NaCl 0,9%):</b>	0.35 ml/min
<b>Particle size:</b>	4.07 $\mu\text{m}$ - 61% < 5 $\mu\text{m}$

---

<b>Air tube length:</b>	1.5 m
<b>Maximum pressure:</b>	2.2 bar
<b>Compressor air flow:</b>	12 L/min
<b>Operating air flow:</b>	5 L/min
<b>Acoustic noise level (Compressor without tube):</b>	55 dBA
<b>Operating cycle:</b>	20 min ON / 40 min OFF
<b>Operating conditions:</b>	10-40 °C; 30-85% maximum relative humidity; 700-1060 hPa atmospheric pressure
<b>Storage and shipping conditions:</b>	-10 - +40 °C; 10-95% maximum relative humidity; 700-1060 hPa atmospheric pressure

<b>Product Weight:</b>	1,300 g
<b>Dimensions:</b>	140 x 137 x 92 mm
<b>Power supply:</b>	Power plug
<b>Expected service life:</b>	1,000 hours
<b>IP Class:</b>	IP21
<b>Reference to standard:</b>	EN13544-1; EN60601-1; EN60601-2; EN60601-1-6; IEC60601-1-11

---

We reserve the right to apply technical changes.

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HONSUN (NANTONG) Co., Ltd
Shanghai International Holding Corp.
GmbH (Europe)